
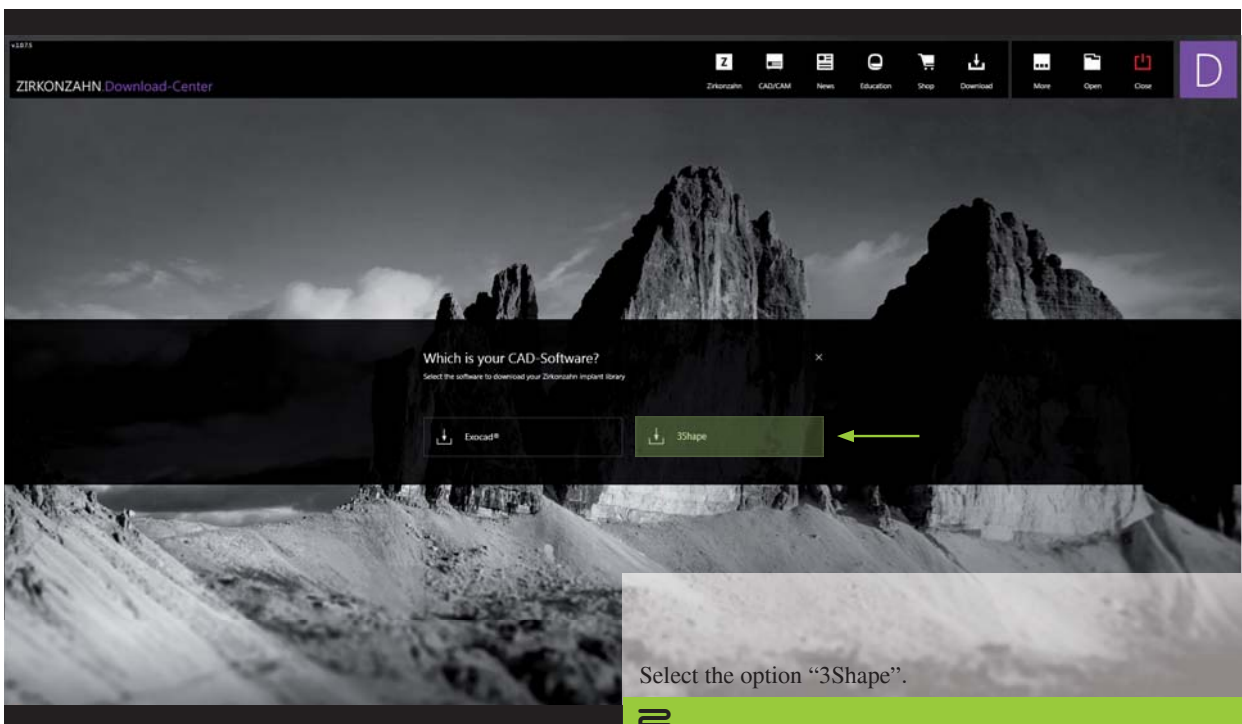




Zirkonzahn.Download-Center Download instructions for 3Shape libraries

Version: 21/02/2019 – Page 1 of 5

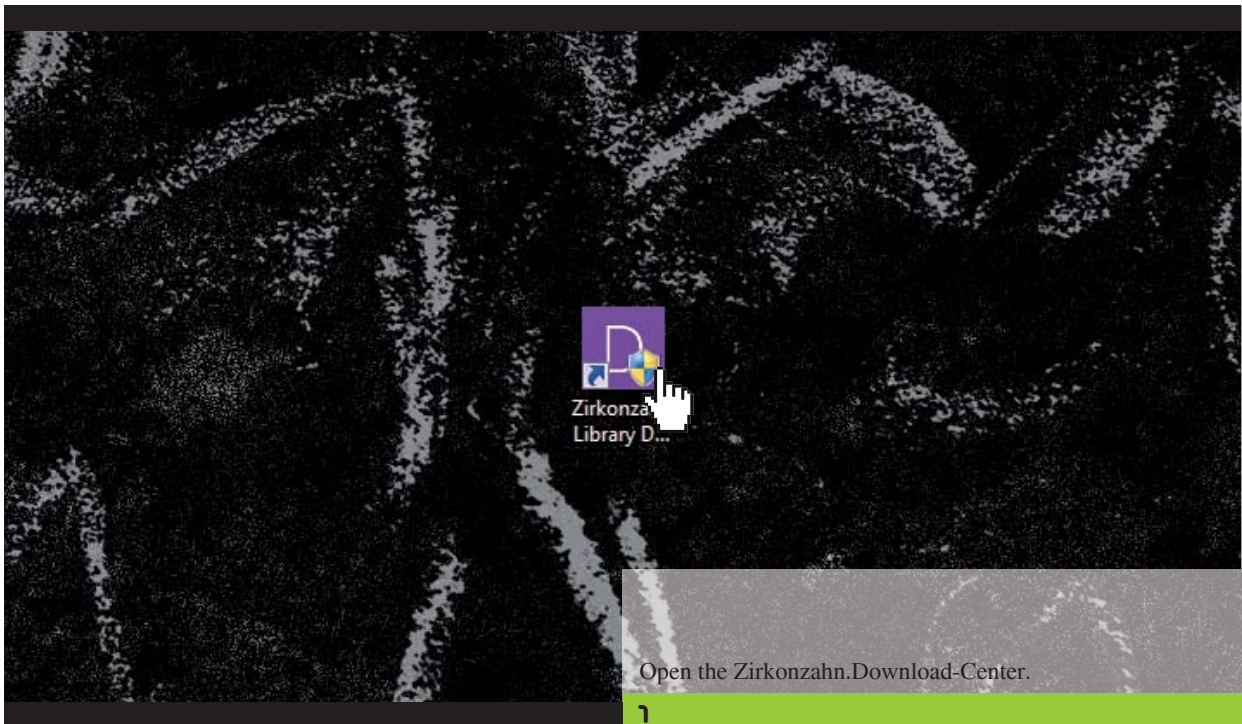


Zirkonzahn.Download-Center Download instructions for 3Shape libraries

Version: 21/02/2019 – Page 2 of 5

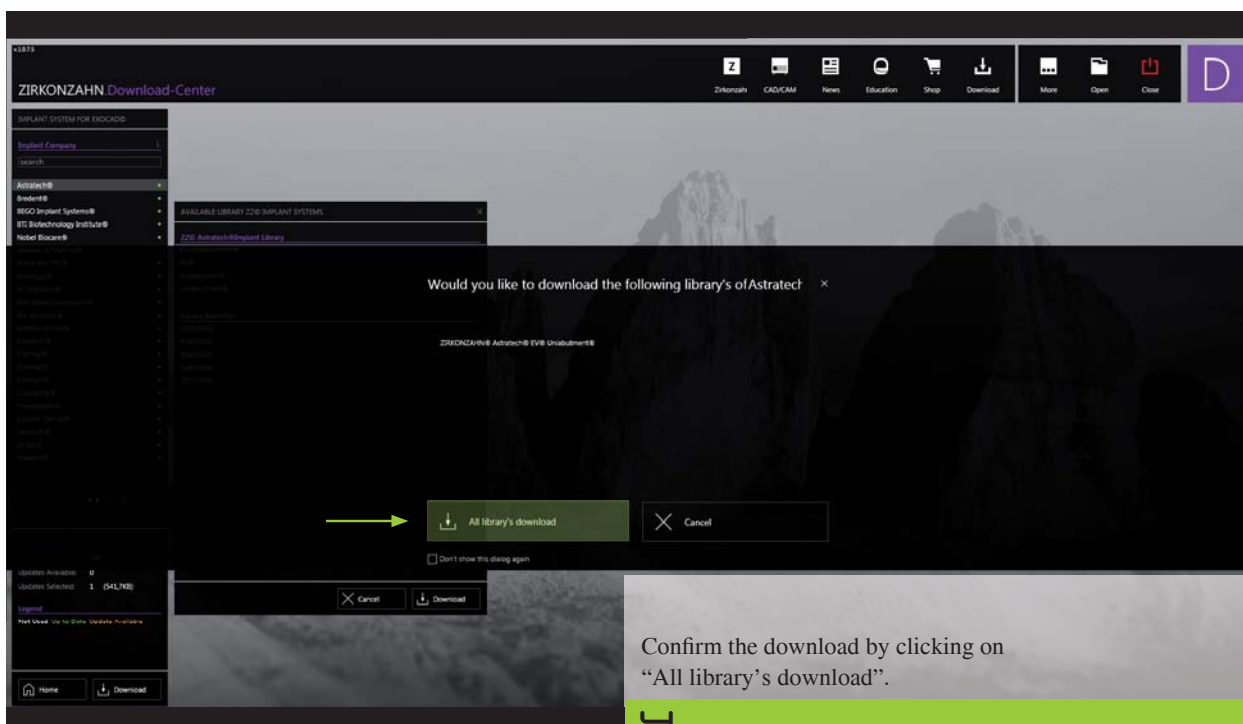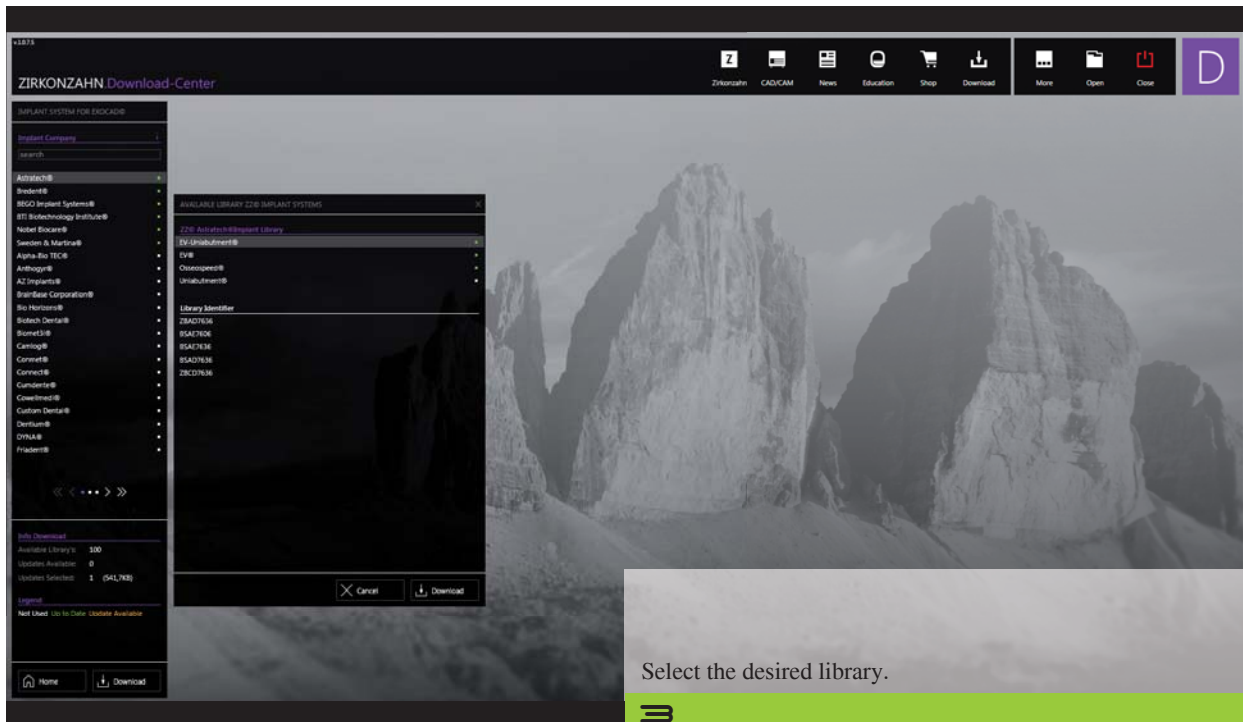


Zirkonzahn.Download-Center Download instructions for 3Shape libraries

Version: 21/02/2019 – Page 3 of 5

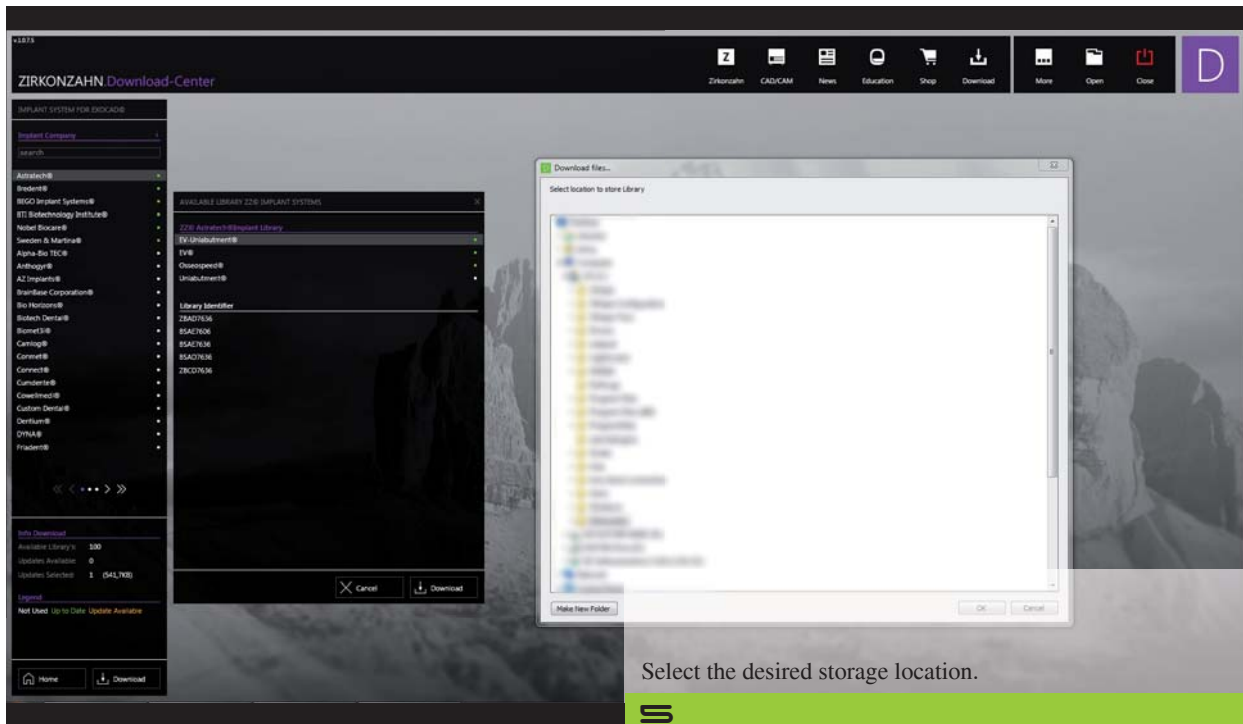

The screenshot shows the Zirkonzahn Download-Center web application. On the left, there is a sidebar with a search bar and a list of implant systems. The main area displays a list of available libraries. A "Download files..." dialog box is open, showing a tree view of the file system for selection. Below the dialog, a green bar contains the number 5.

Select the desired storage location.

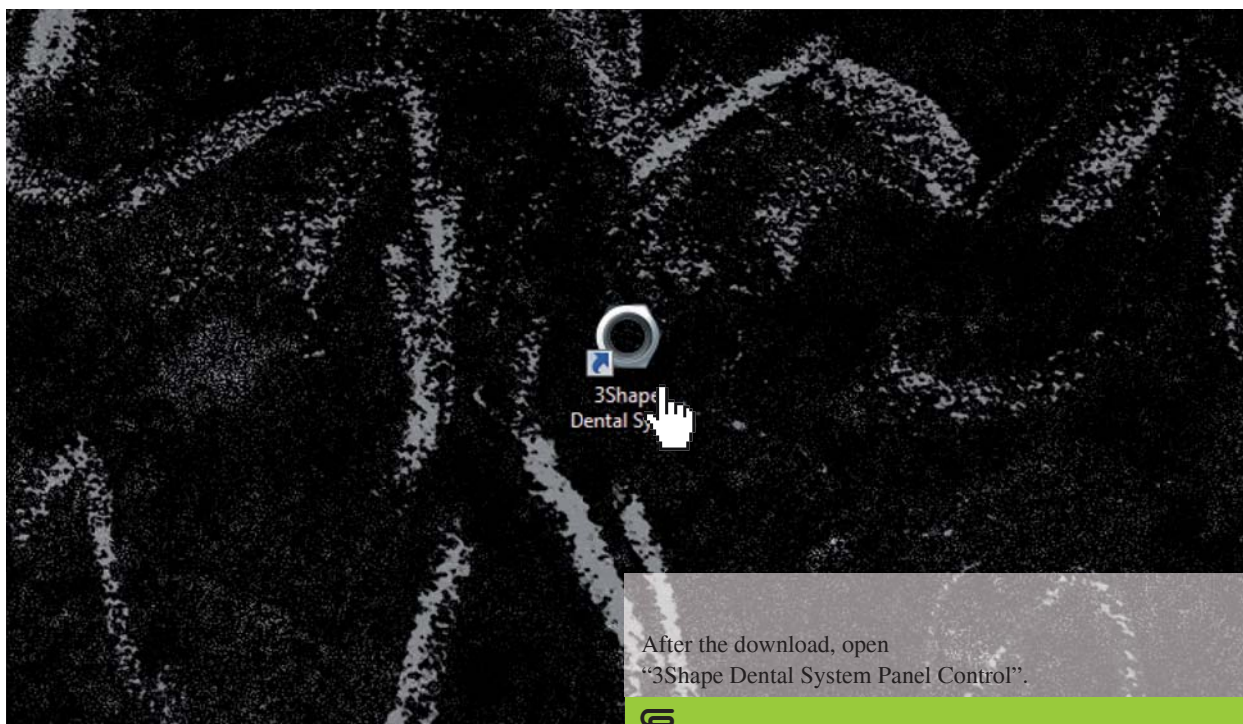


The screenshot shows a dark, textured background with a 3Shape Dental System icon in the center. The icon consists of a white circular shape with a blue square and a white cursor pointing to it. Below the icon, the text "3Shape Dental System" is visible. A green bar at the bottom right contains the number 6.

After the download, open
"3Shape Dental System Panel Control".

Zirkonzahn.Download-Center Download instructions for 3Shape libraries


Version: 21/02/2019 – Page 4 of 5

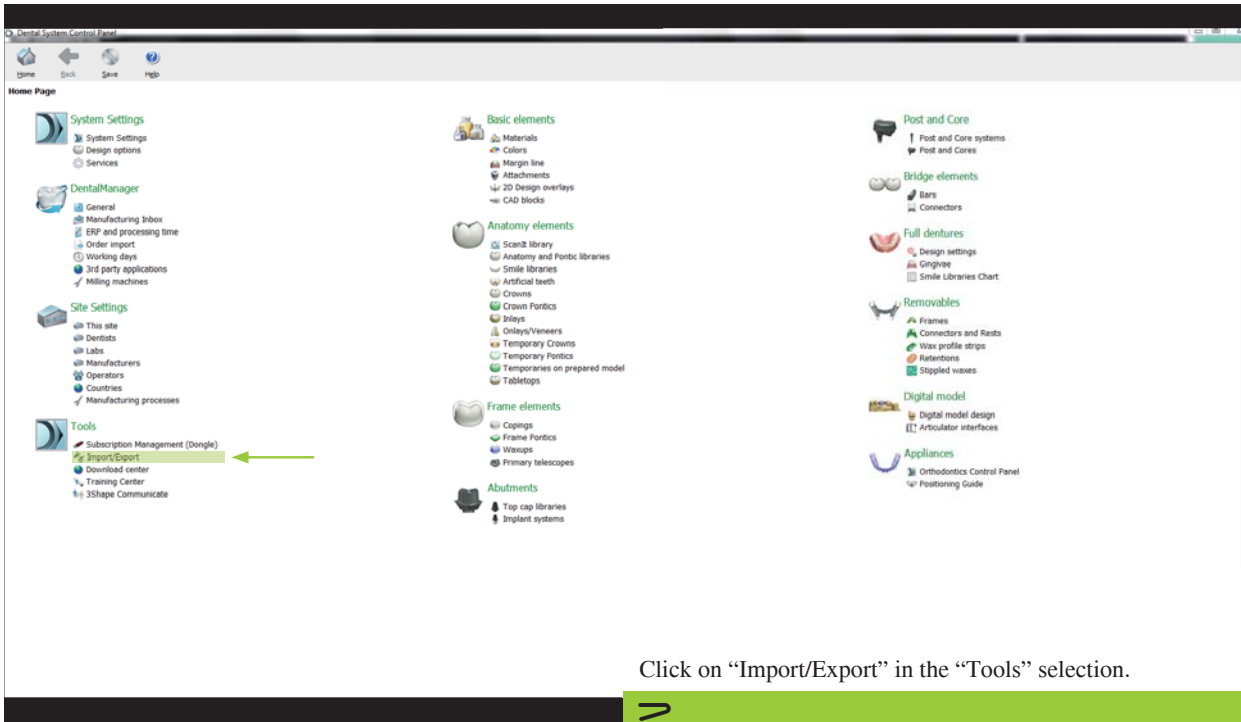

Home Page

- System Settings
 - System Settings
 - Design options
 - Services
- DentalManager
 - General
 - Manufacturing Inbox
 - ERP and processing time
 - Order import
 - Working days
 - 3rd party applications
 - Milling machines
- Site Settings
 - This site
 - Dentists
 - Labs
 - Manufacturers
 - Operators
 - Countries
 - Manufacturing processes
- Tools
 - Subscription Management (Dongle)
 - Import/Export**
 - Download center
 - Training Center
 - 3Shape Communicate
- Basic elements
 - Materials
 - Colors
 - Margin line
 - Attachments
 - 2D Design overlays
 - CAD blocks
- Anatomy elements
 - Scan2 library
 - Anatomy and Pontic libraries
 - Smile libraries
 - Artificial teeth
 - Crowns
 - Crown Pontics
 - Inlays
 - Delays/Veneers
 - Temporary Crowns
 - Temporary Pontics
 - Temporaries on prepared model
 - Tablettops
- Frame elements
 - Copings
 - Frame Pontics
 - Waxups
 - Primary telescopes
- Abutments
 - Top cap libraries
 - Implant systems
- Post and Core
 - Post and Core systems
 - Post and Cores
- Bridge elements
 - Bars
 - Connectors
- Full dentures
 - Design settings
 - Grillvie
 - Smile Libraries Chart
- Removables
 - Frames
 - Connectors and Rests
 - Wax profile strips
 - Retentions
 - Stippled waxes
- Digital model
 - Digital model design
 - Articulator interfaces
- Appliances
 - Orthodontics Control Panel
 - Positioning Guide

Click on "Import/Export" in the "Tools" selection.

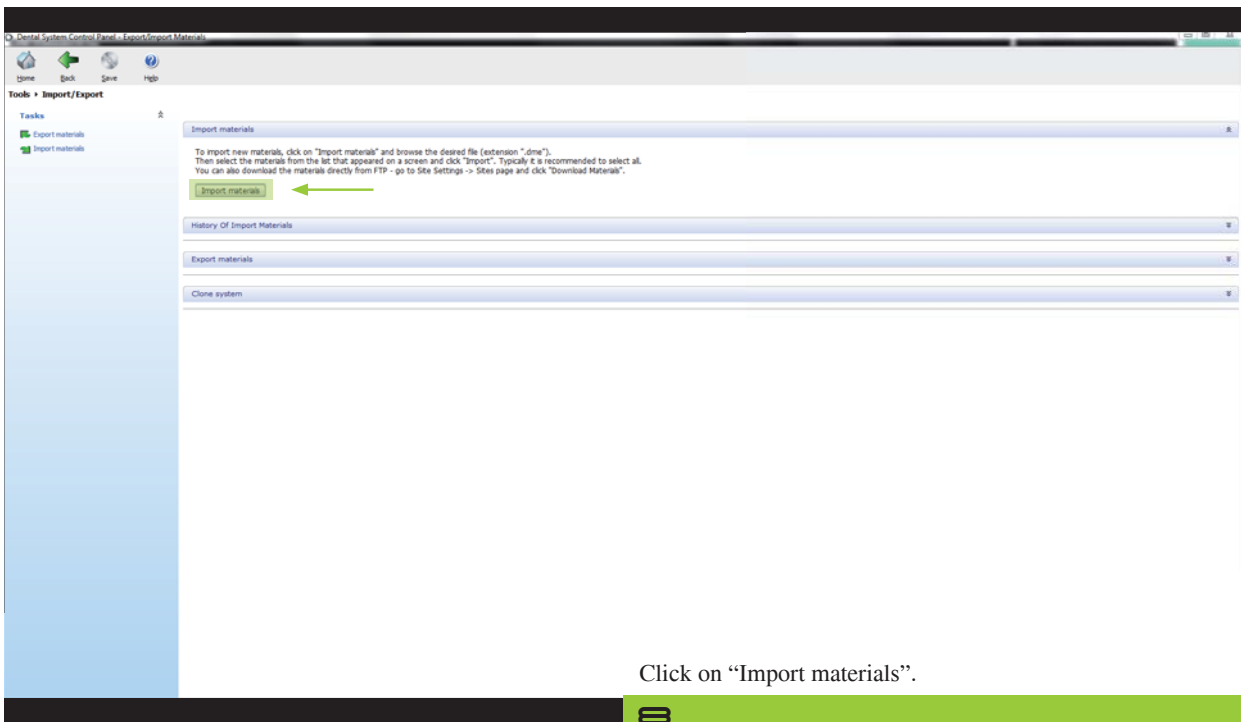

Dental System Control Panel - Import/Export Materials

Tools > Import/Export

Tasks

- Export materials
- Import materials**

Import materials

To import new materials, click on "Import materials" and browse the desired file (extension *.dmg). Then select the materials from the list that appeared on a screen and click "Import". Typically it is recommended to select all. You can also download the materials directly from FTP - go to Site Settings -> Site page and click "Download Materials".

Import materials

History Of Import Materials

Export materials

Clone system

Click on "Import materials".

Zirkonzahn.Download-Center Download instructions for 3Shape libraries

Version: 21/02/2019 – Page 5 of 5

